Rotaid AED Cabinets

SOLID PLUS HEAT

INSTALLATIEHANDLEIDING • GUIDE D'INSTALLATION • MONTAGEANLEITUNG
INSTALLATION MANUAL • MANUALE DI INSTALAZIONE • MANUAL DE
INSTRUCCIONES • INSTRUKCJA MONTAŻU • INSTALLATIONSMANUAL

Parts-list

In the box you will find:

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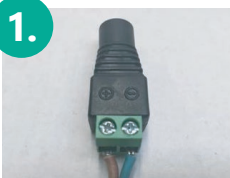
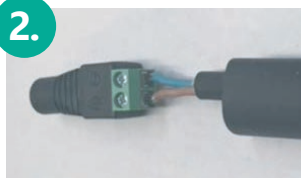
4

ROTAID SOLID PLUS HEAT English instructions

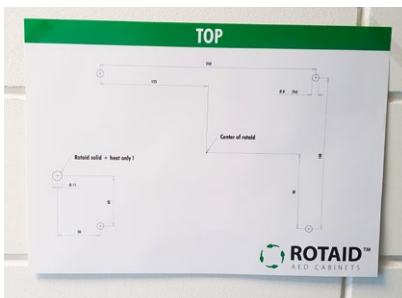
PLEASE NOTE: Only a certified company or electrician should install the cabinet because of an electrical installation. The heating is not ready for use yet!

1. Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

To ensure the defibrillator stays within its operational temperature range during the whole year, the Rotaid should be located in the shade at all times.

2. Use the paper mould to mark the holes for mounting the back plate, 4 in total. Please also mark the additional hole to house the power cable.


3. On the bottom-left of the back plate an eye is located. Use an awl or screwdriver to puncture the sealing plug. Run the whole cable of the heating element through the hole.

Do not attach the heating element to the back plate yet!


4. Attach the back plate by using the four long screws. Next, attach the heating device on the back plate, by using the four small screws and rings. You will find pre-drilled holes in the plate. When correctly placed, the control unit on the heating device is at the bottom left side of the plate.

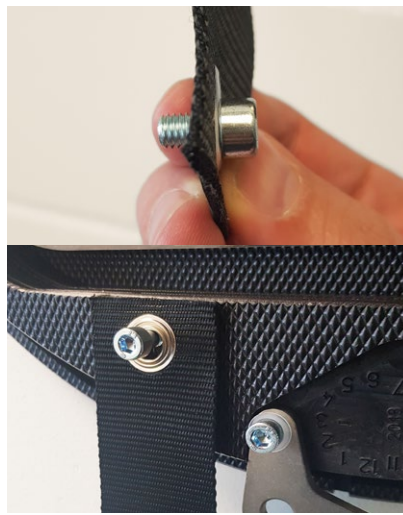5. With the back plate fixed to the wall, now use an Allen key (5mm) to firmly attach the nylon belt to the back plate.

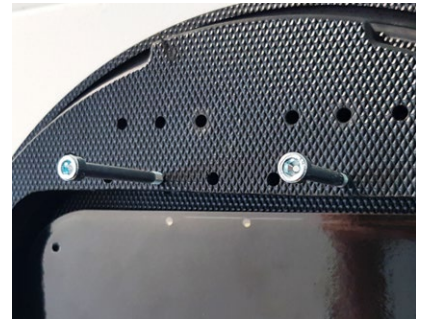6. After attaching the cover and belt, now please use both long bolts to find the perfect position for you AED. Please make sure the bolts are firmly screwed into the block. Place the AED via the bolts by using its bag's handle. The backside of the bag should be placed as close as possible or against to the heating element.

7. You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterclockwise and rotate to the left until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

Continue to **instruction sheet A.**

ROTAID SOLID PLUS HEAT Instruction sheet A.

WATCH OUT! ATTENTION!

Follow the instructions!
*Possible chance of damaging the product.

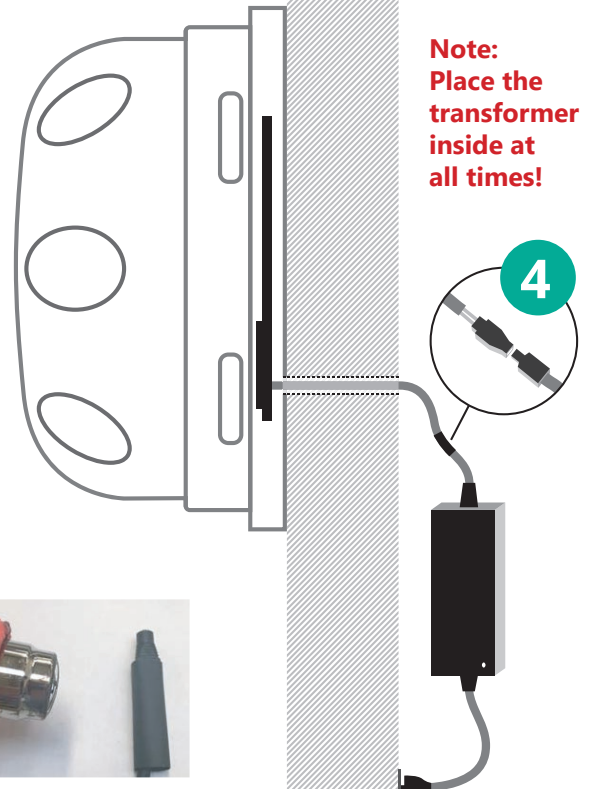
+ Brown - Blue

Attach a connector plug to the cable.

1. Cut the cable to length and strip the outer cable (about 2 centimeters).
2. Remove 5 mm of the brown and blue isolation to unveil the copper wire. Cut off the green-yellow cable completely (if present).
3. Place the crimp sleeve insulator over the connection wire. The cable can also be insulated with tape (after connection).
4. Connect the wires to the connector.
5. Insulate the wires via the sleeve insulator or tape.
6. Now connect the plug to the connector that is attached to the transformer. Plug the power cable into the wall outlet.
7. Immediately after installation, the heating system will go into start-up mode and perform a full system check. Part of this check is that the heater will go to its maximum output for a limited time. As a result of this, the heating mat will become hot. When the system check is successfully finished, the LEDs will be on constantly and the system will go into stand-by mode and its surface temperature will drop. When the system check fails, the LEDs will start to blink. You can then contact your Rotaid partner for further assistance.

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