# ROTAID™

A E D C A B I N E T S

## ROTAID SOLID PLUS

INSTALLATIEHANDLEIDING • GUIDE D'INSTALLATION • INSTALLATION MANUAL  
MONTAGEANLEITUNG • MANUALE DI INSTALAZIONE • MANUAL INSTALACIÓN  
INSTRUKCJA MONTAŻU

# PARTS-LIST

## IN THE BOX YOU WILL FIND:

Installation manual

User guide

Drilling Template

HDPE backplate

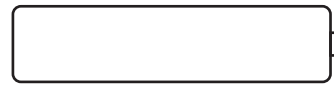
Alarm (if ordered)

Cabinet cover


Screw set

## SCREW SET:

Only for alarmed version


2x AAA Battery

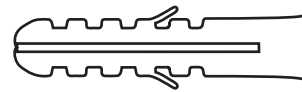4x 8mm Plug

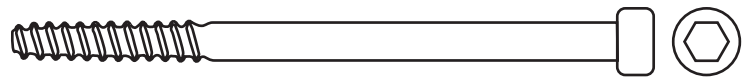
4x Tapping screw 5,5x60

2x DIN screw M6

## Drilling Template

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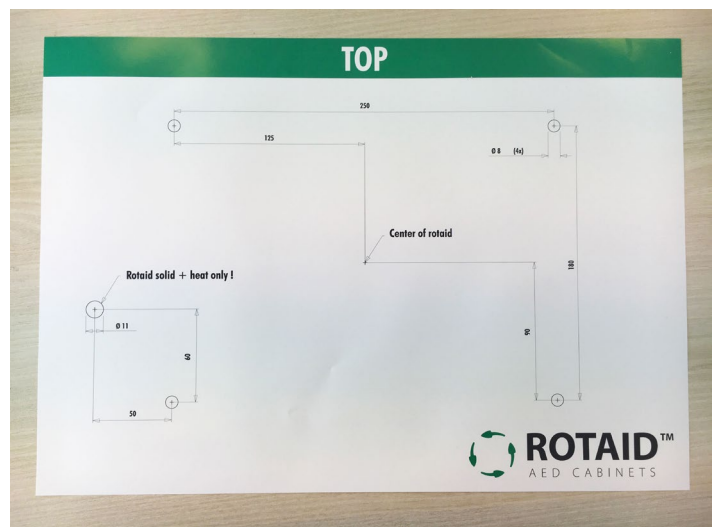Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

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Mark the 4 holes for drilling and use the screws and rings for fixation.

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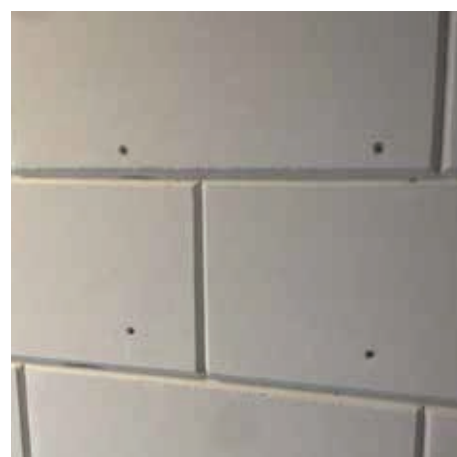
With the back plate fixed to the wall, now use an Allen key (5mm) to firmly attach the nylon belt to the back plate.

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After attaching the cover and belt, now please use both long bolts to find the perfect position for you AED. Please make sure the bolts are firmly screwed into the block.

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You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterclockwise and rotate to the left until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

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Your Rotaïd Solid Plus is now ready for use!

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